REF : M-ZOLABROWN

ZOLA BROWN

Contact

+33 (0)5 32 62 96 60
contact@malys-equipements.com

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Features	Dimensions
Frame color: Natural wood	Length: 190 cm
Equipped with a storage shelf	Width: 80 cm
Adjustable height	Adjustable height: 64 cm to 95 cm
Manual backrest adjustment	Packaging: 194x84x19 cm
Headrest cushion	
Comfortable mattress	
1-year warranty	

Description

The ZOLA BROWN massage table is a professional table with a natural and elegant design, engineered to provide comfort and stability during massage, wellness, and aesthetic treatments.

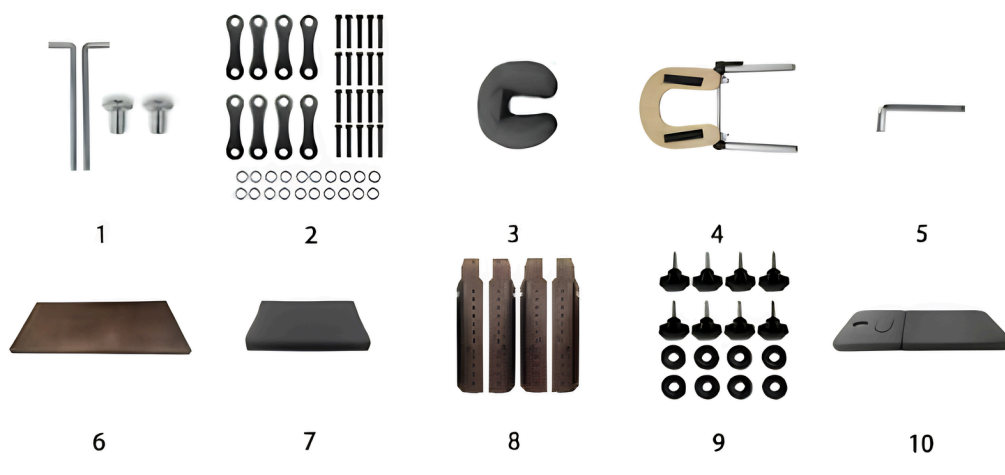
Its solid natural wood frame ensures excellent robustness and a warm finish that blends seamlessly into spas, beauty salons, or treatment clinics. The faux leather upholstery provides a comfortable surface that is easy to clean and maintain.

It comes equipped with a cushion to ensure optimal client comfort while lying down during treatments. The table also features an integrated roll holder, making it convenient to store the linens and accessories needed for your services.

Parts list

Step 1

Place plastic pad (9) onto the five - star handle, and screw it onto bed leg (8) (note : the adjustable part of the bed leg should be level with the bed leg before screwing the handle) and then install the bed leg onto figure 10 bed frame (note : all the nuts on the four bed legs should face inward), and the screws should not be tightened yet.)

Step 2

Install tray (6) using screws (2), fixate shelf screws completely using lever (5).

Step 3

Set the bed in its uprightr position, insert the short side of the backrest rod (1) into the plastic piece on the bed, and insert the long side into the hole of the iron piece on the wooden frame.

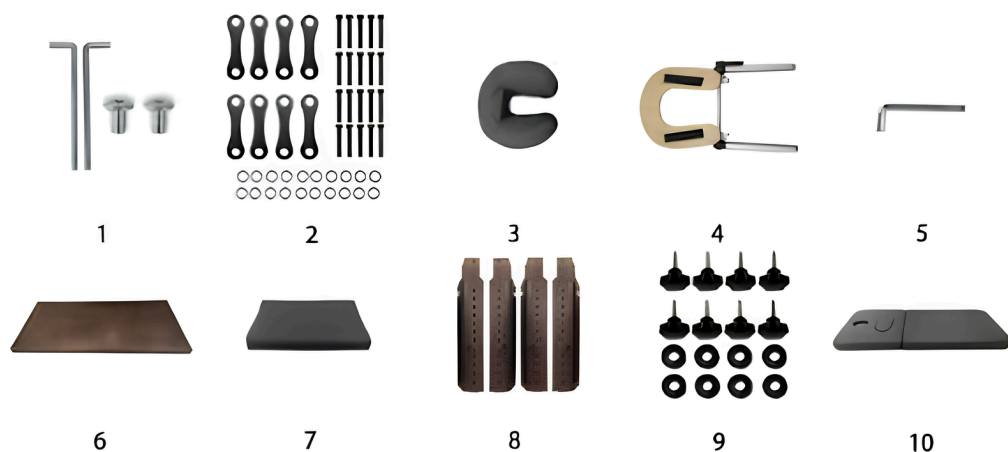
Step 4

Fixate all screws on bed legs completely using wrench (5), fixate face support (3) into its holder (4) and install into bed socket and place head pillow (7) onto the bedsurface.

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