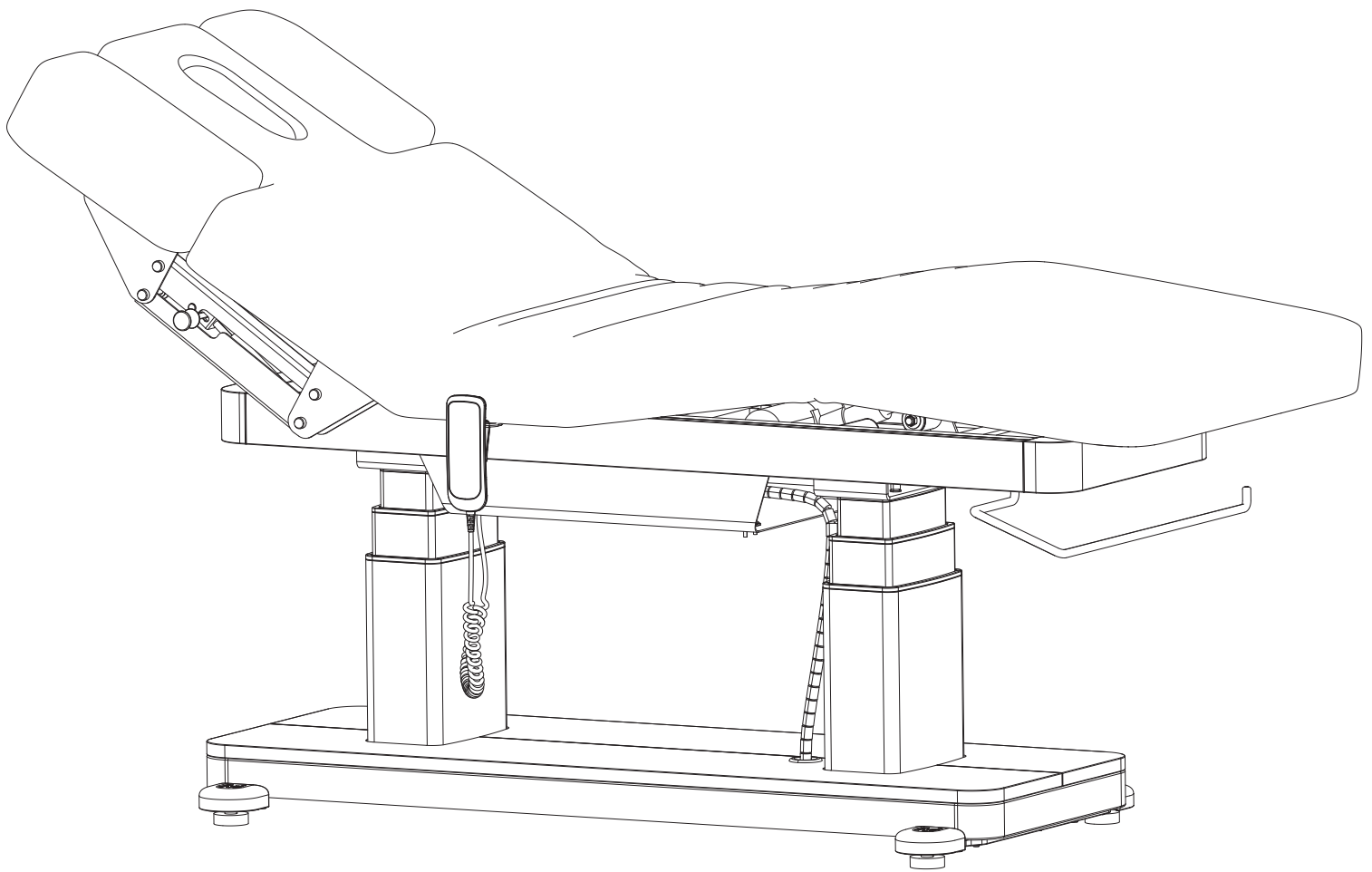
Instrucciones de operación
Operation instruction
Mode d'emploi

VIRGO

Welcome to **VIRGO** electric beauty bed!
Before using this electric beauty bed, please read
the operation instruction carefully.

ES	EN	FR
INDICE	INDICE	INDICE
Composición estructural de la cama de belleza eléctrica	Structural composition of electric beauty bed	Composition structurelle du lit de beauté électrique Pg 3
Ámbito de aplicación	Scope of application	Champ d'application Pg 3
Conexión de instalación	Installation connection	Connexion d'installation Pg 4
Peligro	Warning	Avertissement Pg 5
Diagrama de conexión del sistema de circuitos	Circuit system connection diagram	Schéma de connexion du système de circuit Pg 6
Lista de averías	Malfunction list	Liste des dysfonctionnements Pg 6
Parámetro técnico	Technical parameter	Paramètre technique Pg 7
Mecanismo de ajuste	Adjusting mechanism	Mécanisme d'ajustement Pg 8
Controlador	Controller	Contrôleur Pg 9
Instrucciones de seguridad importantes	Important safety instructions	Consignes de securite importantes Pg 10
Mantenimiento del Producto	Product Maintenance	Entretien du Produit Pg 10
Mantenimiento diario	Daily maintenance	Entretien quotidien Pg 10
Sugerencia de uso	Suggested use	Usage suggéré Pg 11
Limpieza del Producto	Cleaning of the Products	Nettoyage du Produit Pg 11

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EN

PARTS OF ELECTRIC BEAUTY BED

This beauty bed is composed of 4 control programmable control box, motor, hardware, wooden parts, plastic parts, and leather surface. The 4 control programmable control box has the function of connecting the hand controller with 4 motors.

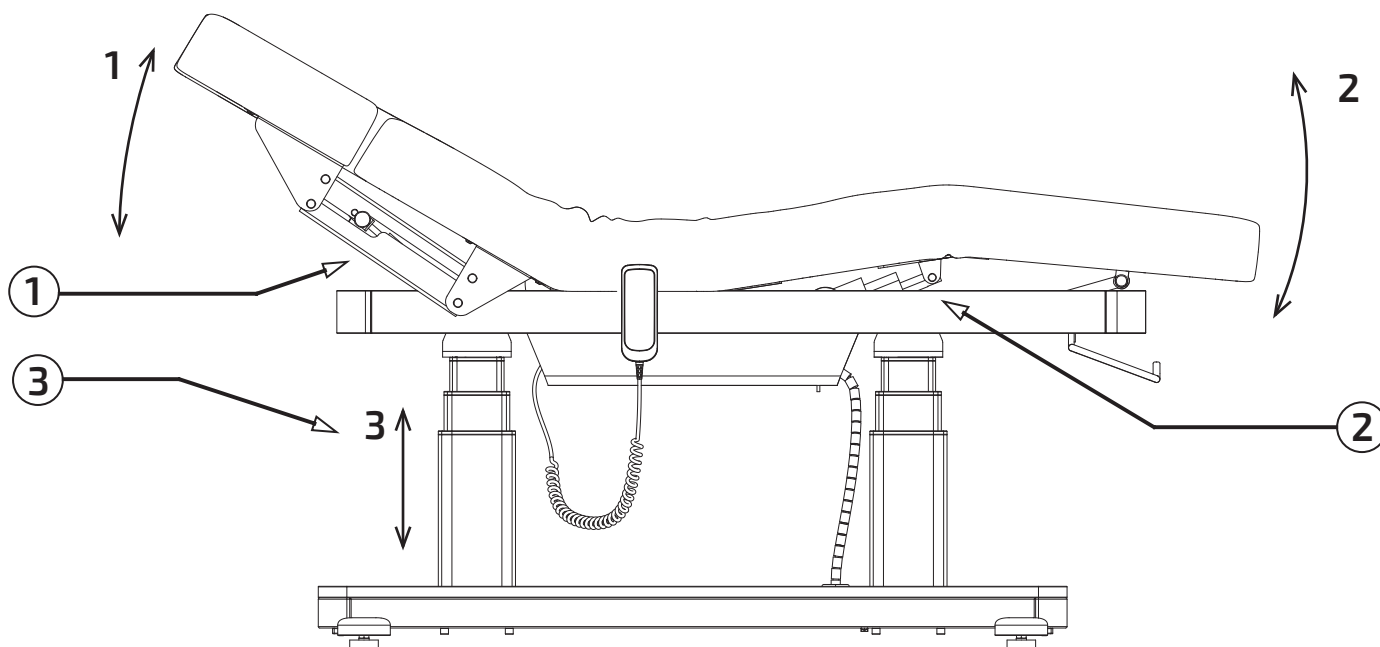
This beauty bed includes a backrest angle control part, a seat pad angle control part, a leg pad control part, and a hight-low control part, which is an integral integrated structure.

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SCOPE OF APPLICATION

This electric beauty bed is suitable for nursing or providing beauty and massage services to customers.

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EN

INSTALLATION CONNECTION

① This machine should be placed in a clean, dry, and ventilated cool place to maintain a good working environment.

② Open box inspection: Open the outer packaging box and check if the entire machine is intact and undamaged. If you have any questions, please immediately inquire with the distributor.

③ Beauty bed installation: First, place the beauty bed in the designated position. The design of this machine has taken into account the stability performance of the entire machine, so there is no need to install anchor fixing screws. The ground that the rubber feet of the beauty bed come into contact with must be level and solid to prevent accidents caused by uneven ground. If the whole machine can rotate or tilt due to uneven ground, a hexagonal open end wrench can be used to adjust the four M10 * 30 rubber feet and M10 nuts under the beauty bed frame to keep it horizontal. When adjusting, the overall level of the machine should be maintained and as many contact pads as possible should be ensured to ensure the stability of the machine.

④ Insert the product power plug correctly into the power box. (See the following figure for details)

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EN

⚠ WARNING

The power must be out off when performing maintenance or cleaning the machine

The plug must be fully inserted into the socket to prevent fire or electric shock.

Do not stand on this equipment.

Keep away from the reach of children. Do not allow children to play with the equipment

Please do not place the equipment in damp places such as bathrooms to avoid electric shock or malfunction; The remote control should also avoid contact with water.

Do not operate this equipment in areas where painting/spraying or oxygen supply is in progress.

Only experienced professionals are allowed to use this equipment

If the plug is damaged, please contact the manufacturer, agent, or skilled personnel for replacement to avoid danger.

During the operation of the beauty bed, ensure that there are no touchable items within the range of movement.

This product is only suitable for use by beauty professionals with professional qualifications.

When pressing the buttons on the hand or foot controller to adjust the backrest angle and the bed height, it is prohibited to place hands, feet, head, and other parts in position of the diagram ① ② ③ (see the following figure for details).

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EN

IMPORTANT SAFETY INSTRUCTIONS

The product's cable cannot be stretched, tightened, or torn, because the product may stop working.

Do not shake the product, do not hit or the product may tip over and fall on users; causing damage to the product and/or injury to the Users.

The voltage of the product is 110v/220v, please choose the appropriate voltage to avoid burning the electrical parts.

When using the product, the maximum current can reach 10A or higher. Use high-power connections.

If possible, use an individual plug, to avoid power failures or fire due to overload.

If the cable is damaged, change it or contact the supplier for after-sales service.

Do not place fuels near the product.

Do not place objects of a different color than the surface, they may stain the skin of the product.

Do not spray water to clean the product, otherwise, it may cause product leakage, short circuits and may lead to electric shock or damage to the product. And that it stops working.

Do not expose the product to the sun, otherwise the color of the product may vary.

Unplug the product when it is not going to be used for a long period of time.

PRODUCT MAINTENANCE

Turn off the power off when product is not in use.

Do not stack sundries above the product, do not use sharp utensils access products.

The product must be placed in the dry environment instead of humid.

The product can not withstand the impact, so do not place heavy objects over 100Kg above the product.

The limit load of the product is 200Kg, using the product in safety load of 150Kg.

When the product does not work, please contact the supplier!

DAILY MAINTENANCE

After each session, use neutral reagents to remove dust and stains from the leather and wooden parts to prevent premature aging of the leather and wooden parts.

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EN

SUGGESTED USE

Before connecting, operating or adjusting this electric beauty bed, carefully read the following instructions and keep this manual safe.

This electric beauty bed uses a three wire power cable. When using it, it must be plugged into a three hole power socket with a ground wire to ensure that the electric beauty bed is properly grounded to prevent accidental electric shock for users, and to ensure the normal operation of the electric beauty bed.

When performing the direct control of the beauty bed's motor lifting operation, the control button must be continuously pressed until the desired position is released.

When using the electric push rod of this electric beauty bed, it should be noted that the frequency of use is Max 6min/h (Continuous operation within 1 hour cannot exceed 6 minutes)

Repairs must be carried out by trained professionals.

After each nursing session, use neutral reagents to remove dust and stains from the skin and decorative parts.

CLEANING OF THE PRODUCTS

Cut off power when cleaning product.

When cleaning product, remember to remove other item in product to prevent slipping down during the cleaning process, resulting in injuries and damaged goods;

Do not spray water to clean the product, otherwise, it may produce a short circuit, or may produce some electric shock.

Use a special cleaning agent when cleaning, oily liquid can not be used, or will cause the leather contamination, corrosion and other phenomena.

Do not use cleaning agent with a liquid dye when cleaning, or will cause the damaged to the local color of the leather;

Do not use cleaning agent with corrosive liquid, or will cause the damaged to the plastic and leather.

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EN

Our products are designed and manufactured according to the applicable regulations.

INFORMATION

Read all instructions before beginning assembly. Keep the instructions for future use.

WARNING

Use only as intended. Do not attempt to stand or kneel on the item. Do not stack. Only flat combinations.

ASSEMBLY

See assembly instructions.

CARE AND CLEANING

Clean the unit with a damp cloth or sponge using warm water and mild soap. Dry thoroughly. Do not use abrasive cleaners, bleach, chemical solvents, or strong detergents.

WARRANTY

2-year limited warranty.

MAINTENANCE

Periodic maintenance tasks should be performed by checking the condition of the product. Ensure the product is in good condition and functioning properly. Check for breaks regularly. Remove the product from service when any condition that could be considered unsafe occurs.

With this guide, you can keep your products in the best possible condition and extend their lifespan. Good luck with your salon maintenance!

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For possible future updates to the document, visit this QR link to see the latest version

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